

Translucidez	Tamanho	Cor	Anel	Temperatura Inicial	Taxa de calor	Temperatura Final	Manutenção	Vácuo On	Vácuo Off
HT		A1, A2, A3, A3.5, B1, B2	100g	700 °C	60 °C /min.	900 °C	20 min.	700 °C	900 °C
		W1, W2, W3, W4				910 °C			910 °C
		A1, A2, A3, A3.5, B1, B2	200g	700 °C	60 °C /min.	910 °C	40 min.	700 °C	910 °C
		W1, W2, W3, W4				920 °C			920 °C
LT		A1, A2, A3, A3.5, B1, B2, B3, B4	100g	700 °C	60 °C /min.	890 °C	20 min.	700 °C	890 °C
		W1, W2, W3, W4, C1, C2, C3, C4, D2, D3, D4				905 °C			905 °C
		A1, A2, A3, A3.5, B1, B2, B3, B4	200g	700 °C	60 °C /min.	900 °C	40 min.	700 °C	900 °C
		W1, W2, W3, W4, C1, C2, C3, C4, D2, D3, D4				910 °C			910 °C
MO		MO0, MO1, MO2, MO3, MO4	100g	700 °C	60 °C /min.	915 °C	20 min.	700 °C	915 °C
		MO0, MO1, MO2, MO3, MO4	200g	700 °C	60 °C /min.	920 °C	40 min.	700 °C	920 °C